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product: **Copran Monolith Colorize Fluid**

product type: marker liquids for colouring liquids

product form: liquids in different shades

selected users: educated users who manufacture zirconium restorations by MAD/MAM or CAD/ CAM milling systems and would like to colour them before sintering.

Specifications:

- Copran Monolith Colorize Fluid
- is a special marker liquid for staining all kinds of zirconium colouring liquids for a better distinction of different areas while staining
 - burns out residue free while sintering
 - is available in following colours: red, blue, green, yellow, orange, brown
 - is water based, does not contain acid, does not corrode metal instruments or metal workbench surfaces
 - does not corrode the heating elements in the sintering furnace
 - does not contain oily substances and infiltrates quick and deep into the structure
 - can easily be dissolved with distilled water

Instructions for use:

add 1ml Copran Monolith Colorize Fluid to a 15ml bottle of colouring liquid using the enclosed syringe – shake well and process liquid as usual. Do not overdose as otherwise the colouring liquid gets too much diluted.

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manufacturer: Whitepeaks Dental Solutions GmbH & Co. KG