CopraTemp

Basis UDI-DI: ++EWPDCCKLL

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CopraTemp

Basis UDI-DI: ++EWPDCCKLL

Manufacturer	Whitepeaks Dental Solutions GmbH Alfredstr. 81 - 45130 Essen – Germany SRN: DE-MF-000021120
Product/ Product type	PMMA blanks for temporary or permanent restorations and splints
Product form	Discs in different sizes and colours
Material type	Polymethyl methacrylate (PMMA) - polymer-based crown and veneering material type 2 class 1 according to DIN EN ISO 10477 - medical device class IIa
Circle of users	Instructed users, who produce individual dental restorations/ splints

Indication/ intended use

CopraTemp is exclusively suitable for the fabrication of dental products:
Temporary or permanent dentures, as well as splints. CopraTemp is certified for use in the oral cavity and can remain in the mouth for a short time or permanently.

CopraTemp is a medical device intended for the fabrication of dental prostheses/splints intended for temporary or long-term use, which are partially introduced into the human body by means of a clinical intervention, but in this case are inserted into the teeth or attached to the tooth structure by means of cement and are therefore classified as class IIa.

CopraTemp A1, A2, A3, B1 and CopraTemp Symphony A1, A2, A3, B1

The tooth-coloured products CopraTemp A1, A2, A3, B1 and CopraTemp Symphony A1, A2, A3, B1 are suitable for the fabrication of temporary crowns, bridges, inlays, onlays, etc., which are later replaced by the final restoration. However, permanent retention in the oral cavity as a permanent restoration is also possible.

CopraTemp Clear

CopraTemp Clear is colourless and is used for the fabrication of splints. Permanent retention in the oral cavity is indicated.

CopraTemp Base C + V

The pink-coloured product CopraTemp Base is suitable for the fabrication of denture bases, which are later replaced by the final denture. However, permanent retention in the oral cavity as a permanent restoration is also possible.

Contraindication

Do not use in case of proven hypersensitivity against one of the components.

Gebrauchsanweisung/ Instructions for use/ Manuel d'instruction/ Manuale d'istruzioni/ Manual de instrucciones/ Instrukcja obsługi/ Bruksanvisning/ Instruksjoner for bruk/ Käyttöohjeet/ Gebruiksaanwijzing/ Εγχειρίδιο οδηγιών/ Upute za korištenje/ Manual de instrucțiuni/ Instrukcija/ Návod k použití/ Návod na použitie/ Használati utasítás

CopraTemp

Basis UDI-DI: ++EWPDCkTLL

Material properties/ technical data

material composition: CopraTemp A1, A2, A3, B1, CopraTemp Symphony A1, A2, A3, B1 CopraTemp Base	Polymethyl methacrylate, colour pigments.
material composition: CopraTemp Clear	Polymethylmethacrylate
flexural strength (Weibull)	> 100 MPa
water absorption	~ 20 bis 23 µg/mm ³
solubility	< 1 µg/mm ³
Vickers hardness	> 20 HV
modulus of elasticity	> 2500 MPa
residual monomer	< 1%

Instructions for use

Burs for acrylic materials are recommended. Polishing is possible with polishing paste and soft hair brushes. Pay attention not to overheat the material.

In the occlusal region the minimum wall thickness must be 1,0 mm and 0,6 mm in the cervical region. The connector profile has to be minimum 10mm² in anterior region and minimum 15mm² in the posterior region. In the posterior region max. two pontics are allowed.

Veneering:

All standard veneering materials, that are suitable for PMMA

Safety instructions

Warning: Always wear respiratory protection (filter class FFP2), tightly fitting safety goggles, protective gloves and protective clothing and switch on suction device. The dust produced during processing of this product may cause irritation to skin/ eyes/ respiratory system. Avoid inhalation/ contact with skin, mouth and eyes. Do not eat or drink while working. Keep away from food and beverages. Wash hands after use. Remove contaminated clothing and protective equipment before entering areas where food will be eaten. Keep away from sources of ignition. Do not smoke.

Storage

Protect from heat and direct sunlight. Store in the original packaging.

Disposal

Dispose of product and packaging in accordance with local/ regional/ national/ international regulations.

Notice

Any serious incident, that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

The summary report on safety and clinical performance according to (EU) MDR 2017 / 745 Article 32 (SSCP) is available in the European Database for Medicinal Products (Eudamed) under the associated base UDI-DI as soon as the corresponding database module exists.

<https://ec.europa.eu/tools/eudamed>

Gebrauchsanweisung/ Instructions for use/ Manuel d'instruction/ Manuale d'istruzioni/ Manual de instrucciones/
Instrukcja obsługi/ Bruksanvisning/ Instruksjoner for bruk/ Käyttöohjeet/ Gebruiksaanwijzing/ Εγχειρίδιο οδηγιών/
Upute za korištenje/ Manual de instrucțiuni/ Instrukcija/ Návod k použití/ Návod na použitie/ Használati utasítás

CopraTemp

Basis UDI-DI: ++EWPDCkTLL

Explanation of the markings on the packaging

Symbol for „article number“

Symbol for „LOT number“

Confirmation: The product complies with the applicable European directives.

Symbol for „number of products in package“

Symbol for „follow the instructions for use“

Symbol for „is a medical device“

Symbol for “production date”

Rx only

Symbol for “Caution: US Federal law restricts this device to sale by or on the order of a licensed physician or dentist.”

Gebrauchsanweisung/ Instructions for use/ Manuel d'instruction/ Manuale d'istruzioni/ Manual de instrucciones/
Instrukcja obsługi/ Bruksanvisning/ Instruksjoner for bruk/ Käyttöohjeet/ Gebruiksaanwijzing/ Εγχειρίδιο οδηγιών/
Upute za korištenje/ Manual de instruțiuni/ Instrukcija/ Návod k použití/ Návod na použitie/ Használati utasítás

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